# Simport® Scientific® Histotainer™ - Instructions for use

EN

All Histotainers™ M960 and M961 series are made from a polypropylene jar and a polyethylene cap. These are half-pre-filled with 10% Neutral Buffered Formalin.

## Intended use

These containers are designed for the collection, transport and storage of histology specimens. [Refer to the SDS for safety recommendations when manipulating formalin.](#)

## Storage

These products should be stored at room temperature in upright position.

## INSTRUCTIONS FOR USE

Before use:

1. Verify if packaged product has been kept in a horizontal position in its original packaging.
2. Ensure that formalin is present at half of the container's volume.

Directions for use:

1. Complete the patient identification label
2. Place container on a flat counter surface.
3. Open the Container with care to prevent spillage.
4. Keep lid near the work environment.
5. Place biopsy in the pre-identified container.
6. Close and tighten the lid on container with a clockwise motion until you reach a tight closure.
7. Process the samples according to specific laboratory needs using the same precautions as above.

## Warning

Not regulated for Air transportation. Dispose of contents/container to an approved waste and/or biohazard disposal plant.

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Simport® Scientific Inc., 2588, Bernard-Pilon, Beloeil (Québec) J3G 4S5 Canada