M430 & M530 Series

Superflo Cassettes

The **SuperFlo™** Cassette, a new generation emerging in the Simport® Histology Family.

Molded from a special high-density polymer, these cassettes keep specimens safely submerged in liquid and are resistant to the chemical action of most histological solvents.

Lateral vent design for a maximum fluid exchange and efficient tissue processing. Sloped walls, creating a basket for small tissue management.

Available in loose, sleeved, or taped packaging for easy printer loading.



A family owned 🌞 Canadian company

simport.com

The SuperFlo™ pore design allows a steady flow of fluid for maximum reagent exchange. Such a high level of fluid exchange results in less incidence of unfixed or under-processed tissues and faster fixation/processing of tissue specimens.

The cassettes have a 45-degree writing surface and are designed for optimal workflow in the most popular Cassette Printers.

The **SuperFlo™** can be opened and closed as required and they always relock securely without danger of specimen loss.





M530 Series for Routine

Opening of 0.91 x 3.67 mm








M531 Series for Biopsy

Opening of 0.64 x 0.64 mm

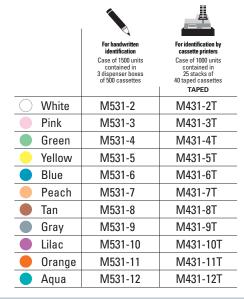





M532 Series for Micro Biopsy

Opening of 0.30 x 0.30 mm

ĺ		
	For handwritten identification Case of 1500 units contained in 3 dispenser boxes of 500 cassettes	For identification by cassette printers Case of 1000 units contained in 25 stacks of 40 taped cassettes TAPED
White	M532-2	M432-2T
Pink	M532-3	M432-3T
Green	M532-4	M432-4T
Yellow	M532-5	M432-5T
Blue	M532-6	M432-6T
Peach	M532-7	M432-7T
Tan	M532-8	M432-8T
Gray	M532-9	M432-9T
Lilac	M532-10	M432-10T
Orange	M532-11	M432-11T
Aqua	M532-12	M432-12T

SuperFlo™ One-Piece Model

This model is a one-piece molded cassette so no separate lids to manage. It can be opened and closed up to five times guaranteed, securely relocking to avoid specimen loss.




back of the base in an open position.

holding it by the front tab.


click is heard.

To open, tilt cover backward. It will remain attached to the base and can be closed again.

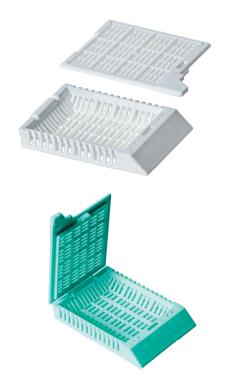

M533 Series Cassettes for Routine


Opening of 0.91 x 3.67 mm

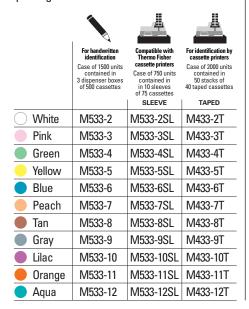




M534 Series Cassettes for Biopsy


Opening of 0.64 x 0.64 mm

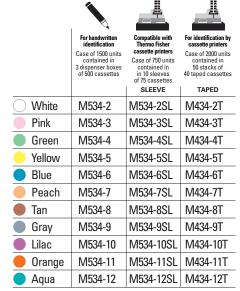

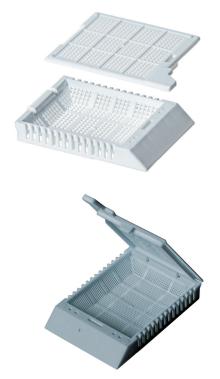


M535 Series for Micro Biopsy Cassettes


Opening of 0.30 x 0.30 mm

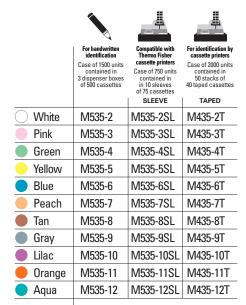


SuperFlo™ Hinged-Hook Model

This model has a special hinge that holds the base and the cover together. The hinge design allows the cassette to be opened and closed as often as required.

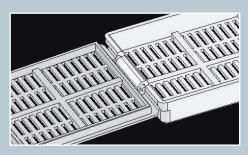

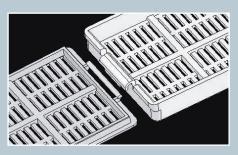

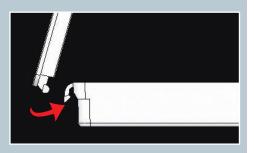


Simport M430 & M530

SuperFLO[™] Cassettes




Routine

Opening of 0.91 x 3.67 mm

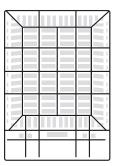


Actual size

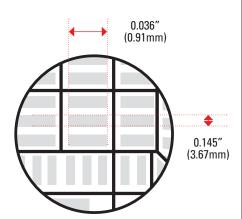




Biopsy

Opening of 0.64 x 0.64 mm

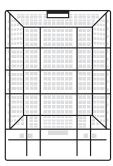


Actual size

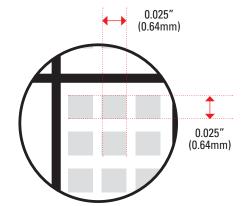



Micro Biopsy

Opening of 0.30 x 0.30 mm

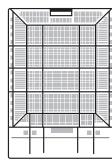

Actual size

